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EN. Instructions for using the MATERNI Single Use FlowCup®.

General information.

Materni always recommends breastfeeding in the first instance.

Intended use:

The Materni FlowCup® is intended for the administration of breast milk, a suitable liquid formula or an appropriate Oral Rehydration Solution (ORS). The Materni FlowCup® is a suitable medical device for Oral Rehydration Therapy (ORT). In the context of the clinical indication "mild to moderate dehydration" in infants and young children, the Materni FlowCup® is a device for applying a suitable Oral Rehydration Therapy. Always consult a specialist before applying Oral Rehydration Therapy. The single-use Materni FlowCup® is ready for immediate use and is hygienically sealed in individual packaging.

For the safety and health of your child - WARNING.

Never leave the packaging and parts unattended. Keep out of reach of children. Only use the Materni FlowCup® for its intended purpose as described above. Inspect each component before each use. If there are any signs of damage, wear or weakness, do not use the product and discard it immediately. If the packaging is visibly damaged or has been opened, or if the FlowCup® is damaged, do not use the Materni FlowCup®. The Materni FlowCup® may not be reused. Re-use or recurring use creates a risk of infection, cross-infection and loss of mechanical, chemical or microbiological properties. Always use the FlowCup® in combination with Materni accessories. Always use the Materni FlowCup® under the supervision of a specialist. Always check the temperature of the liquid. The FlowCup® may not be heated.

Packaging and storage:

The Materni FlowCup® is individually packaged and ready for use. Do not touch the inside of the FlowCup® and the inside of the ring before use. Always store the Materni FlowCup® in a clean, dark, dry and cool place.

Assembly:

Open the packaging of the FlowCup® on the ring side. Hold the FlowCup® by the soft part (cup) while it is still protected by the closed side of the packaging. Place the FlowCup® on the bottle and turn the ring tight. Make sure that the FlowCup® ring is attached securely enough to the bottle's screw thread to prevent any leakage. Now remove all the packaging. The Materni FlowCup® is now ready for use.

Use:

Disinfect your hands thoroughly with soap and water or according to local regulations before touching the FlowCup®. Make sure the liquid is at the desired temperature before administering. Hold the FlowCup® like a normal feeding bottle, with the bottom of the bottle tilting slightly upwards. Touch the lip with the soft part of the cup to trigger the rooting reflex. Place the tip of the FlowCup® on the baby's lower lip and gently squeeze the sides of the cup (see picture). The harder you squeeze, the faster the liquid flows out of the bottle into the reservoir. Adjust the flow rate to the baby's drinking behavior. The baby will automatically lick up the liquid from the cup. In some cases the baby will "suck" the tip of the cup instead of licking. This depends on the child, but both drinking methods are fine.

Ring material: polyethylene

Cup material: TPE

Ethylene oxide (EO), latex, phthalates and bisphenol A-free.

Disposal: in accordance with local regulations.

Complies with the European Medical Device Regulation 2017/745.

Complies with Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food. Complies with standard EN 14350: Child use and care articles - Drinking equipment. Manufactured in Belgium. In event of malfunction, negative result, undesirable or adverse reactions, stop using the device and contact your physician and the manufacturer. Date of latest revision of instruction for use: 25/7/2023.

CBM (Clean by Manufacturing): This product is hygienic and can be used safely without being cleaned first.

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