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EN. Instructions for using the MATERNI Single Use FlowSpoon®.

General information.
 Materni always recommends breastfeeding in the first instance.

Intended use:
 The Materni FlowSpoon® is intended for the collection of breast milk, in particular colostrum, during manual expression and for the administration of breast milk or an appropriate Oral Rehydration Solution (ORS). The Materni FlowSpoon® is a suitable medical device for Oral Rehydration Therapy (ORT). In the context of the clinical indication "mild to moderate dehydration" in infants and young children, the Materni FlowSpoon® is a device for applying a suitable course of Oral Rehydration Therapy. Always consult a specialist before applying a course of Oral Rehydration Therapy.
 The single-use Materni FlowSpoon® is ready for immediate use and is hygienically sealed in individual packaging.

For the safety and health of your child - WARNING.
 Never leave the packaging and parts unattended. Keep out of reach of children.
 Only use the Materni FlowSpoon® for its intended purpose as described above.
 Inspect each component before each use. If there are any signs of damage, wear or weakness, do not use the product and discard it immediately. If the packaging is visibly damaged or has been opened, or if the FlowSpoon® is damaged, do not use the Materni FlowSpoon®.
 The Materni FlowSpoon® may not be reused. Re-use or recurring use creates a risk of infection, cross-infection and loss of mechanical, chemical or microbiological properties. Always use the Materni FlowSpoon® under the supervision of a specialist. Always check the temperature of the liquid.
 The FlowSpoon® may not be heated.

Packaging and storage:
 The Materni FlowSpoon® is individually packaged and ready for use. Do not touch the inside of the FlowSpoon® before use. Always store the Materni FlowSpoon® in a clean, dark, dry and cool place.

Assembly:
 Open the packaging of the FlowSpoon® on the handle side. Grasp the handle and then remove the rest of the packaging. In doing this, avoid contact with the inside of the FlowSpoon®. The Materni FlowSpoon® is now ready for use.

Use:
 Disinfect your hands thoroughly with soap and water or according to local regulations before touching the FlowSpoon®.
 Materni recommends the manual expression of colostrum. To do this, hold the FlowSpoon® under the nipple as a reservoir for milk collection. Feed the baby as soon as possible after expressing to ensure optimal hygiene and milk temperature. For Oral Rehydration Therapy, fill the FlowSpoon® with the rehydration fluid. Hold the FlowSpoon® by the handle and touch the baby's lip with the tip to trigger the rooting reflex. Place the tip of the FlowSpoon® on the baby's lower lip and allow the liquid to flow gently to its lips by holding the FlowSpoon® slightly tilted. Adjust the flow rate to the baby's drinking behavior. The baby will automatically lick up the liquid. In some cases the baby will "suck" the tip of the FlowSpoon® instead of licking. This depends on the child, but both drinking methods are fine.

FlowSpoon® material: TPE
 Ethylene oxide (EO), latex, phthalates and bisphenol A Free.
 Disposal: in accordance with local regulations.
 Complies with the European Medical Device Regulation 2017/745.
 Complies with Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food.
 Complies with standard EN 14350: Child use and care articles - Drinking equipment
 Manufactured in Belgium. In event of malfunction, negative result, undesirable or adverse reactions, stop using the device and contact your physician and the manufacturer. Date of latest revision of instruction for use: 25/7/2023

CBM (Clean by Manufacturing):
 This product is hygienic and can be used safely without being cleaned first.

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