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EN. Instructions for using the MATERNI Single Use Teat.

General information.

Materni always recommends breastfeeding in the first instance.

Intended use:

The Materni teat is intended for the administration of breast milk, a suitable liquid formula or an appropriate Oral Rehydration Solution (ORS).

The Materni teat is a suitable medical device for Oral Rehydration Therapy (ORT). In the context of the clinical indication "mild to moderate dehydration" in infants and young children, the Materni teat is a device for applying a suitable course of Oral Rehydration Therapy. Always consult a specialist before applying a course of Oral Rehydration Therapy. The single-use Materni teat is ready for immediate use and is hygienically sealed in individual packaging.

For the safety and health of your child - WARNING.

Never leave the packaging and parts unattended. Keep out of reach of children.

Only use the Materni teat for its intended purpose as described above.

Inspect each component before each use. If there are any signs of damage, wear or weakness, do not use the product and discard it immediately. If the packaging is visibly damaged or has been opened, or if the Materni teat is damaged, do not use the Materni teat.

The Materni teat may not be reused. Re-use or recurring use creates a risk of infection, cross-infection and loss of mechanical, chemical or microbiological properties. Always use the teat in combination with Materni accessories. Always use the Materni teat under the supervision of a specialist. Always check the temperature of the liquid. Never use the Materni teat as a soother. The teat may not be heated.

Packaging and storage:

The Materni teat is individually packaged and ready for use. Do not touch the soft part of the teat or the inside of the ring prior to use. Always store the Materni teat in a clean, dark, dry and cool place.

Assembly:

Open the packaging of the teat on the ring side. Hold the teat by the soft part while it is still protected by the closed side of the packaging. Place the teat on the bottle and turn the ring tight. Make sure that the teat ring is attached securely enough to the bottle's screw thread to prevent any leakage. Do not, however, overtighten the teat on the bottle. This may hinder drinking since no air can enter the bottle, thereby creating a vacuum. Now remove all the packaging. The Materni teat is now ready for use.

Use:

Disinfect your hands thoroughly with soap and water or according to local regulations before touching the teat. Make sure the liquid is at the desired temperature before administering. Hold the Materni bottle and teat inclined, with the bottom of the bottle tilting slightly upwards. Touch the teat to the baby's lip to trigger the rooting reflex. Observe if the tongue is pointing downwards when the baby takes the bottle. Present the bottle carefully and adjust the flow rate. Do this by rotating the bottle in such a way that the desired speed, indicated by a circle or number, is directly opposite the baby's nose (position 1 = slowest speed). Adjust the flow rate to the baby's drinking behavior. Remember that the baby must be fed at a steady pace and incorporate an occasional break during feeding so the baby can burp.

Ring material: polypropylene

Teat material: TPE

Ethylene oxide (EO), latex, phthalates and bisphenol A Free.

Disposal: in accordance with local regulations.

Complies with the European Medical Device Regulation 2017/745.

Complies with Regulation (EU) No 10/2011 on plastic materials and articles intended to come into contact with food.

Complies with standard EN 14350: Child use and care articles - Drinking equipment

Manufactured in Belgium.

In case of malfunction, negative result, undesirable or adverse reactions, stop using the device and contact your physician and the manufacturer.

Date of latest revision of instruction for use: 07/5/2025

CBM (Clean by Manufacturing):
This product is hygienic and can be used safely without being cleaned first.

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