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KLINIKA Medical GmbH, Am Dornbusch 29, 61267 Usingen, Germany

REF 101200100

KLINIMARK Wire breast localization needle (double hook)

INSTRUCTION FOR USE:

The KLINIMARK Wire breast localization needle is used for preoperative marking of suspicious nonpalpable lesions to facilitate the location by the surgeon. The procedure described herein is provided for informational purposes. The user must ascertain the terms of use, based on their clinical experience and the type of intervention to be made. KLINIKA Medical GmbH disclaims any liability for injury caused as a result of tampering with, or incorrect use of the device.

1. Prior to use, make sure that the package is not opened or damaged and check the sterilization date.
2. Open the package.
3. Disinfect the insertion point and administer the local anaesthetic.
4. Optionally, make a small incision with a scalpel to facilitate passage of the needle through the upper layers of skin.
5. Remove protective cover from the needle and the metal marker.
6. Regulate the position of the sliding stopper according to the depth to be reached.

Before puncturing, the curved portions of the wire must be completely withdrawn into the cannula. The wire is withdrawn into the cannula until both markings are visible behind the handle of the cannula.

7. Using imaging equipment, such as ultrasound or mammogram, puncture the target area with the cannula.
8. Hold the marker in place and remove the cannula. The hook is completely out of the needle tip when the black mark printed on the back of the marker, near the handle, is no longer visible. Treat the wound and fix the wire in an appropriate way (adhesive tape, gauze).
9. After the treatment, please ensure the appropriate disposal of the cannula in the proper cannula container.

Storage Conditions

The product must not be used if the package is unsealed, damaged or wet. Keep in a clean, dry place and avoid exposure to direct sunlight and high temperature. Sterility is guaranteed only if unopened. Verify the integrity of the packaging and the validity of the product before use.

Warnings and precautions:

This product has been explicitly designed and manufactured for single use only. The product may not be reused. A safe use is no longer guaranteed. The product is not designed for cleaning and sterilization processes after use. The sterility of reprocessed single-use product, is therefore not guaranteed. The risk of unintentional injuries and infections, especially cross-infection with a patient and medical personnel increases immeasurably. The company KLINIKA Medical GmbH accepts no responsibility for the consequences of re-sterilisation.

Contraindications:

All contraindications for the respective areas of application, apply as they are understood for cannulas of this type in accordance with current regulations.

WARNING THIS NEEDLE IS NOT MRI (magnetic resonance imaging)-compatible!